

HICIDE S-5

Concentrated surface disinfectant for healthcare centers


HICIDE S-5 is dilutable, one-step disinfectant cleaner formulated with proprietary hydrogen peroxide technology. Besides, HICIDE S-5 has excellent compatibility with most materials used in healthcare centers. It is recommended for routine cleaning and disinfection of hard surfaces.

Characteristics

- One-step disinfectant cleaner
- Environmentally responsible-active ingredients break down to oxygen and water
- Excellent cleaning performance
- Without causing antimicrobial resistance
- Cost-effective

Product Specification

Color/Form	Colorless to pale yellow clear solution
pH	0.8- 1.8
Scent	Characteristic odor
Shelf Life	18 months
Solubility (in water)	Completely soluble

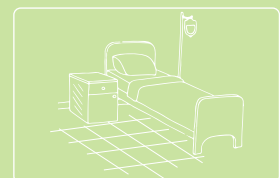

THIS IS A HISEPT PRODUCT, WITH
**HIGH-LEVEL
SAFETY &
PROTECTION**

Ingredients

- 100 g HICIDE S-5 contains:
- 4.25 g Hydrogen peroxide

Auxiliary ingredients:

- Surfactants
- Stabilizer
- Corrosion inhibitors
- Deionized water

Microbiological Effectiveness

Meets European Standards

Bactericidal Activity

According to EN 13727 Staphylococcus. aureus/ ATCC 6538
(contact time: 10 minutes use dilution: 2%) Enterococcus. hirae/ ATCC 10541
Pseudomonas. aeruginosa/ ATCC 15442

According to EN 13697 Staphylococcus. aureus/ ATCC 6538
(contact time: 30 minutes use dilution: 2%) Enterococcus. hirae/ ATCC 10541
Pseudomonas. aeruginosa/ ATCC 15442
Escherichia. coli/ ATCC 10536

Yeasticidal Activity

According to EN 13624 Candida. albicans/ ATCC 10231
(contact time: 15 minutes use dilution: 6%)

According to EN 13697 Candida. albicans/ ATCC 10231
(contact time: 30 minutes use dilution: 6%)

Fungicidal Activity

According to EN 13624 Aspergillus. niger/ ATCC 16404
(contact time: 30 minutes use dilution: 6%)

According to EN 13697 Aspergillus. niger/ ATCC 16404
(contact time: 30 minutes use dilution: 6%)

Directions for use

Dilute HICIDE S-5 with water according to below:

2 %- Bactericidal and yeasticidal

6 %- Fungicidal

1 %- Non-food contact sanitizer or general purpose cleaning

0.5 %- Floor and general purpose cleaner

Routine recommendation: 2 % in 30 minutes

Note: Rinsing is not necessary unless floors are coated with finish or restorer. All food contact surfaces must be rinsed with potable water. Do not use on glassware, utensils or dishes. Prior to first use on a particular surface, it is recommended to test in a small, discreet area if there is concern of incompatibility. Not recommended for floors at 6 %.

Compatibility

HICIDE S-5 is compatible with most materials such as stainless steel, plastic, poly propylene, copper, PVC and silicon.


2/2 Latest revision: Jan 1, 2023

Packagings

3 canisters per carton,	5 L each
6 canisters per carton,	2.5 L each
6 bottles per carton,	1 L each

Hazard Rating

Concentrate	NFPA
Health	2
Flammability	0
Reactivity	0

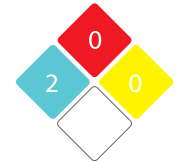

Warning

- Keep out of reach of children and unaware person.
- Store between +5 °C to + 35 °C and out of direct sunlight.



Certifications

- >> ISO 9001: 2015
- >> ISO 13485: 2016
- >> GMP
- >> FSC



HISEPT[®]
HIGH-LEVEL SAFETY & PROTECTION

Middle East Office
Behban Pharmed Lotus Co.
Address : Unit 113, 12 Floor, Bldg. B, Kaveh
Complex, Azadi Ave., Tehran, Iran
Tel: (+9821) 66581294-5/66581269

info@hisept.com

World Locations: Düsseldorf, Kuala Lumpur, Istanbul, Tehran, Baghdad