


MEDICLEAN 1616

Alkaline detergent for reprocessing surgical instruments

HISEPT Mediclean 1616 is a highly concentrated and efficient alkaline detergent, specially formulated for automated cleaning of surgical instruments. This product is recommended for both heat-sensitive and heat-resistant instruments. It is usable in various automated washer disinfectors.

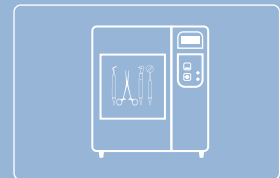

Characteristics

- Without any residues
- Usable with soft or hard water
- Excellent material compatibility
- Non-foaming
- Strong cleaning performance

Product Specification

Color/Form	Colorless to pale yellow clear solution
pH (0.5 %)	12 - 13
Scent	Odorless
Shelf Life	24 months



THIS IS A HISEPT PRODUCT, WITH
**HIGH-LEVEL
SAFETY &
PROTECTION**

Ingredients

HISEPT MEDICLEAN 1616 contains:

- Sodium silicate
- Alkaline agent
- Complexing agent
- Deionized water

Microbiological Specification

According to	
USP 40	Bacterial count <10 CFU/10g
	Yeast & mold total count < 10 CFU/10g
	No growth of Staphylococcus. aureus
	No growth of Pseudomonas. aeruginosa
	No growth of Escherichia. coli
	No growth of Fungi



Directions for use

This product should be diluted based on the washing machine instructions. Please refer to the manufacturer's recommendations.

Compatibility

HISEPT MEDICLEAN 1616 is compatible with most materials such as stainless steel, copper, plastic, rubber and silicon.


2/2 Latest revision: May 1, 2022

Packagings

2 canisters per carton,	10 L each
3 canisters per carton,	5 L each

Hazard Rating

Concentrate	NFPA
Health	3
Flammability	0
Reactivity	0

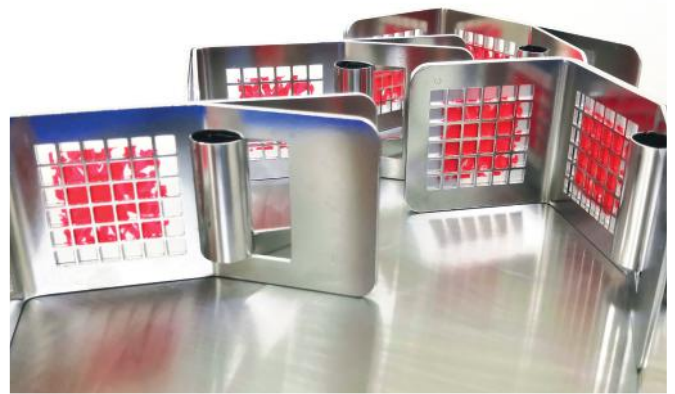

Warning

- Wear personal protective equipment such as work gloves, boots, mask, and work clothes during application and dilution.
- In case of contact with skin and eyes, rinse immediately with plenty of water.
- Keep in the original container and sealed.
- Keep away from combustible materials and acidic compounds.
- Keep out of reach of children.
- Store in a well-ventilated area.



Certifications

- >> ISO 9001: 2015
- >> ISO 13485: 2016
- >> GMP
- >> FSC



HISEPT[®]
HIGH-LEVEL SAFETY & PROTECTION

Middle East Office
Behban Pharmed Lotus Co.
Address : Unit 113, 12 Floor, Bldg. B, Kaveh
Complex, Azadi Ave., Tehran, Iran
Tel: (+9821) 66581294-5/66581269

info@hisept.com

World Locations: Düsseldorf, Kuala Lumpur, Istanbul, Tehran, Baghdad